

# Three Layer Non-Medical Protection Mask

## Applied Industrial Technologies and Lentus



### Features and Benefits

- Three layer construction
  - » Top and bottom layers are constructed from 25GSM spunbound non-woven polypropylene and pigment
  - » Middle layer is constructed from 25GSM meltblown non-woven polypropylene
- Three pleats of folds to allow the user to expand the mask so it covers the area from the nose to the chin
- Elasticized ear loop to secure mask behind the ears
- Three ply, fluid resistant
- Fiberglass and latex free
- Length is approximately 175 mm and width is approximately 95 mm
- 50 single use masks per box

Item No.	Mfr. No.
117226903	MSAF_24795200

Product of:



**Applied® has partnered with Lentus and their Handy Antibac brand to bring you non-medical personal protection masks that are made from three layers of soft non-woven material with a flexible nose strip and elastic ear loops.** They're suitable for personal protection use both at home or work and can be used indoors and out. This comfortable and non-irritating mask helps filter air pollution, dust, smoke, germs and pollen.

### Instructions for Use:

Prior to placing the mask on your face, wash your hands thoroughly. Place the colored side of the mask facing outwards with the flexible nose strip on top and mold to fit the bridge of your nose. Hold the mask by the ear loops when fitting and place one loop over each ear, one at a time. Pull the bottom of the mask down to fully cover your mouth. Try to avoid touching the mask once on your face. If the mask becomes soiled or damp, replace it with a new one.

### Removal of Mask:

Prior to removing the mask, wash your hands thoroughly. Try to avoid touching the mask itself and use the ear loops to remove from your face and dispose of in trash bin. Wash your hands thoroughly after removing.

**WARNING** – This mask should not be used for toxic or hazardous air contaminants, such as paint spray, asbestos, silica, cotton, or other toxic dusts, gases, fumes and vapors. It is to be used for personal protection only and is not intended for medical purposes. Replace or discontinue using mask if breathing becomes difficult. This mask is not to be used except as indicated.

Contact Your Local Applied® Service Center  
**1-877-279-2799**

Authorized distributor:



Corporate Headquarters  
 1 Applied Plaza  
 Cleveland, Ohio 44115  
 Toll Free Phone: 1-877-279-2799  
[Applied.com](http://Applied.com)

DISCLAIMER: The above content is provided by the supplier. Applied® does not guarantee the accuracy or timeliness of the content. The above content is provided "AS IS", AND ANY AND ALL EXPRESS AND IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS ARE HEREBY EXCLUDED. All warranties, if any, are solely from the supplier.