

# Hospital-Grade Disinfectant

Applied Industrial Technologies and Tri-Chem



**Applied® is proud to offer CLUBZ HOSPITAL QUAT-CLEANER from Tri-Chem.** This powerful cleaner kills 99.9% of germs and bacteria and has demonstrated its effectiveness against viruses similar to COVID-19 on hard, non-porous surfaces. CLUBZ is also highly effective against bacteria, fungi, mold and mildew and is specifically designed for use in hospitals, food processing plants, commercial kitchens, restrooms, and other public facilities.

## Features and Benefits

- General ready-to-use disinfectant, cleaner, tuberculocidal
- Kills 99.9% of bacteria
- Kills 99.9% of germs
- Cuts grease and grime
- Eliminates odors
- Kills Pandemic 2009 H1N1 influenza A virus
- Meets OSHA Bloodborne Pathogen standard
- For HIV, HBV and HCV
- Doesn't contain phosphates or phosphorous

## How to Use

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

This product is not for use on critical or semi-critical medical device surfaces.

Fill or refill the 16 oz. CLUBZ HOSPITAL QUAT-CLEANER spray bottle with CLUBZ HOSPITAL QUAT-CLEANER refill. Pour product into spray bottle over sink or tub basin.

Item No.	Mfr. No.
117189384	CHEM_CLUBZ CASE

Product of:



Authorized distributor:



# Tri-Chem Technical Data Sheet

## DISINFECTION, DEODORIZING AND CLEANING

Remove gross filth or heavy soil prior to application of the product. Hold container six to eight inches from surface to be treated. Spray area until it is covered with the solution. Allow product to penetrate and the surface to remain wet for three minutes (for Bloodborne Pathogens: allow surface to remain wet for one minute for HIV 1, HBV and HCV). No scrubbing is necessary. Wipe off with a clean cloth, mop or sponge. The product will not leave grit or soap scum.

**To Disinfect Food Processing Premises:** Before using CLUBZ HOSPITAL QUAT-CLEANER, food products and packaging materials must be removed or carefully protected. For floors, walls and storage areas, hold container six to eight inches from the surface to be treated and spray area until it is covered with the solution. Allow CLUBZ HOSPITAL QUAT-CLEANER to penetrate and remain wet for 3 minutes (for Bloodborne Pathogens: allow surface to remain wet for one minute for HIV 1, HBV and HCV). Wipe off with a clean cloth, mop or sponge and thoroughly rinse all surfaces in the area with potable water. Heavily soiled areas must be pre-cleaned before using this product.

**To Disinfect Food Service Establishment Food Contact Surfaces:** Before using CLUBZ HOSPITAL QUAT-CLEANER food products and packaging materials must be removed or carefully protected. For countertops, appliances, tables, hold container six to eight inches from the surface to be treated and spray area until it is covered with the solution. Allow CLUBZ HOSPITAL QUAT-CLEANER to penetrate and remain wet for 3 minutes (for Bloodborne Pathogens: allow surface to remain wet for one minute for HIV 1, HBV and HCV). Wipe off with a clean cloth, mop or sponge and thoroughly rinse all surfaces in the area with potable water. Heavily soiled areas must be pre-cleaned before using this product. Do not use on eating utensils, glassware and cookware.

## PERSONAL PROTECTION

When handling items soiled with blood or bodily fluids use disposable latex gloves, gowns, masks and eye coverings.

## CLEANING PROCEDURES

Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this product.

## CONTACT TIME

Allow surface to remain wet for one minute for HIV 1, HBV and HVC. These contact times will not control other common types of viruses and bacteria.

## STORAGE AND DISPOSAL

**Pesticide Storage:** Store in a dry place no lower in temperature than 50°F or higher than 120°F.

**Pesticide Disposal:** Pesticide wastes are acutely hazardous. Improper disposal of excess pesticide, spray mixture, or rinsate is a violation of Federal Law. If these wastes cannot be disposed of according to label instructions, contact your State Pesticide, Environmental Control Agency, or the Hazardous Waste representative at the nearest EPA Regional Office for guidance.

**Container Disposal:** Nonrefillable container. Do not reuse or refill container. Clean container promptly after emptying.

**Triple Rinse as Follows:** Fill container ¼ full with water and recap. Agitate vigorously. Follow pesticide disposal instructions for rinsate disposal. Drain for 10 seconds after the flow begins to drip. Repeat procedure two more times. Offer for recycling or reconditioning, if available. If not available, puncture and dispose in a sanitary landfill.

**Container Disposal for Refill Spray Bottle:** Do not reuse empty spray bottle except with refill bottle, otherwise discard in trash. Do not reuse empty refill bottle. Wrap refill bottle and discard in trash. Offer for recycling or reconditioning, if available.

Contact Your Local Applied® Service Center  
**1-877-279-2799**

DISCLAIMER: The above content is provided by the supplier. Applied® does not guarantee the accuracy or timeliness of the content. The above content is provided "AS IS", AND ANY AND ALL EXPRESS AND IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS ARE HEREBY EXCLUDED. All warranties, if any, are solely from the supplier.

Authorized distributor:



Corporate Headquarters  
1 Applied Plaza  
Cleveland, Ohio 44115  
Toll Free Phone: 1-877-279-2799  
[Applied.com](http://Applied.com)